

CT Scan Protocol
SMART Fusion

X-Ray Lab Technician:

The following details are required with completing the CT scan for the Nobel Clinician software. If you have any questions about the information listed, please contact Ryan Haupt at 714-529-9792 prior to completing the scan(s).

1. The CT device should be set to scan at a minimum of 10 seconds at .4 voxel slices for both the patient scan and appliance by itself scan.
2. Separate the upper and lower arches with a cotton roll or other non-metallic radiolucent material.
3. Provide the dicom data only on a CDROM or email through a large file format service (hightail.com/dropbox.com) to ryan@hauptlab.com

If a CDROM is to be created, please mail a copy of the CDROM to the doctor and Haupt Dental Lab

Haupt Dental Lab
1220 E. Birch Street Suite 201
Brea, CA 92821
(714) 529-9792