

## Verification Index – Made in the Mouth

1. Remove temporary denture (if applicable)
2. Screw the implant impression copings to the implants.
3. Lute the impression copings together **in the mouth** using floss to act as a matrix to “tie” the impression copings together
4. Using a light cure resin, build a bridge on the floss connecting the impression copings together.
5. Remove the index from the mouth and screw implant replicas to the cylinders (see photo)
6. Mix a low-expansion die stone (enclosed) a ratio of 21mL water to 100g of powder
7. Use the black crown box enclosed to hold the stone (see photo)
8. Set the analogs/verification index into the wet stone while attached the verification index (see photo)
9. Let the stone set for a minimum of 30minutes to ensure accuracy.
10. Remove the verification index from the analogs and return it to the mouth (screw back on the implants in the mouth)
11. Fit the custom tray over the index to ensure a good fit. Align the screw holes to ensure access.
12. Use the cut pieces of toothpick to “plug” the screw access holes of the verification index
13. Make an impression of the upper verification index in place using care to capture the soft tissue side and borders.
14. Inspect the tissue side “ check for good tissue impression “

If you have any questions, please give us a call!

# VERIFICATION INDEX

